

# BALMORAL II ACTIVE MATTRESS





**Product Code:** 

**Description:** 



#### HAMBALM003

- The mattress is made up of 27 individual cells; 23 alternating air cells and 4 static pillow section cells
- The mattress has an integrated air pocket base
- Cells are made of pure PU
- Pillow section for patient stability
- All cells are interchangeable for ease of repair and cleaning
- 4-way stretch welded PU cover which is vapour permeable and water resistant and has a Crib 5 fire rating
- In the event of a power outage, the Balmoral II system has been designed to protect the end-user.
- The Balmoral II has foam fitted underneath the alternating cells.
- Compatible with Harvest Healthcare's pumps 250 and 3

Mattress Dimensions: 2000 x 900 x 150 mm

Weight: 10kg

Max. Patient Weight: Harvest 250 Pump: 31 Stone (200kg)

Harvest 3 Pump: 40 Stone (254kg)

Pressure Relief Risk: Very High

Pump Technical Specification: Please see separate pump specification sheet

## **Cleaning Guidance:**

Cover should be wiped down with hot water and mild detergent. If heavily soiled, the cover may be machine washed up to 60-95°C and tumble dried. For disinfection use sodium hypochlorite solution max 1 000ppm. Use no bleach, biological or phenolic-based cleaners. Dry completely before storing.

#### Warranty

In terms of Harvest Healthcare Guarantee and Warranties the Harvest Balmoral Plus mattress is guaranteed for three years from date of purchase.

#### **Exclusions:**

• Damage to fabric covers through incorrect use and penetration by sharp instruments is excluded.

### **Supplier Contact Details:**

t: +44(0)1709 377172 e: sales@harvesthealthcare.co.uk w: www.harvesthealthcare.co.uk Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60



Balmoral II cells and foam fitted underneath the alternating cells



## **Easy Care Systems**

Units 5 and 6 Lower Vicarage Road, Southampton, Hampshire, SO19 7RJ

T: 02380 420678

E: info@easycaresystems.co.uk